

# Warranty details

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your local Drive stockist.

## **Drive DeVilbiss Healthcare Ltd**

Sidhil Business Park, Holmfield, Halifax,  
West Yorkshire, HX2 9TN, Great Britain

[info@drivedevilbiss.co.uk](mailto:info@drivedevilbiss.co.uk)

[www.drivedevilbiss.co.uk](http://www.drivedevilbiss.co.uk)

Tel: +44 (0) 845 0600 333

## **Drive DeVilbiss Sidhil Ltd**

4 Trench Road, Mallusk, Newtownabbey,  
BT36 4TY, Northern Ireland



# drive

DeVilbiss<sup>®</sup>  
HEALTHCARE



## Folding Bath Bench

Instruction Manual

## Safety precautions

This folding bath bench is designed for sitting on only, do not stand on it as this could result in failure of the device. Ensure that all the push buttons lock into position and protrude through the holes in the legs. Periodically check all fastenings for tightness. Check the rubber tips for rips, wear & tear, replace if necessary. Always sit in the centre of the bench, do not lean on the edges of the bench as this can cause instability. Do not exceed the maximum weight limit of 135kg (21 stone). The bench is only suitable for one user at a time.

Caution! Ensure that the seat is locked onto the frame and that all the legs are set at the same height, with the push buttons secure, before using the bath bench.

## Assembly


1. Remove product from carton and read all instructions carefully before attempting to assemble the bath bench.
2. Turn seat section upside down. Unfold legs and locate the frame into the metal receptacle on the underside of the seat. Turn seat right way up and press down on seat, ensuring that the frame is fully secured in position.
3. The rear and front leg extensions are marked with a 1 & 2 respectively. Insert leg extensions into the frame, ensuring that the number on the leg and frame corresponds accordingly.



12486KDR

12487KDR

## Height Adjustment

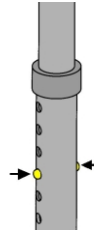

To adjust the leg height:

1. Push in the buttons (as shown).
2. Adjust the section to the required height.
3. When this height is reached, release the buttons but carry on adjusting the section until the buttons lock in position.
4. Repeat for the other 3 legs.

Caution! Ensure that all 4 legs are set to the same height.

## Specification

Product Codes:	12486KDR	12487KDR
Maximum user mass:	135kg (21st)	
Overall Height:	42 - 46cm	64 - 68cm
Overall Width:	43cm	
Overall Depth:	49cm	
Seat (W x D):	43 x 28cm	

Replacement Tip Code: FER22L

## Maintenance

To maintain the quality of the finish, we recommend that the bath bench is wiped down and dried after each use. Periodically, clean the bench with a damp cloth and mild detergent or disinfectant. For example, choride disinfectant (solution of 1000 parts per million) can be used. Do not use an abrasive cleaner or cloth with this product, as damage may occur.

Check for any damage to the seat and/or legs. If in doubt, remove from service.