

User Guide

Boston Home Care Bed – 1202/5/PF/PF

SIDHIL

Introduction

Thank you for purchasing a Sidhil Boston Home Care bed. Key features of your bed include:-

- Maximum user weight of up to 180Kg
- Swivel castors for ease of manoeuvrability and safety.
- Specifically designed, static and dynamic mattresses.
- Full range of bed accessories.
- BioCote – Anti Bacterial coating.



Specifications

1202/5/PF/PF – Boston Home Care Bed			
Maximum Load	180Kg (28 Stone)	Net weight	32Kg (79lb)
Platform height	460mm (18.1")	Under bed clearance	400mm (15.7")
Overall Size	2210 x 940mm (87" x 37")	Mattress area	1980 x 930mm (78" x 36.6")

Assembly & preparing for use

Your bed is designed with fold down legs for easy transport and handling. To assemble your bed a 13mm spanner is required. (The bolt size is M8 which is 13mm across the flats of the bolt head).



WARNING –The bed weight exceeds 25Kg.
Follow appropriate lifting and handling practice.

- 1) Position your bed in the approximate location of use.
- 2) Lifting up the long side of the frame, stand the bed upright on its side.
- 3) Remove the 4 bolts near to each leg fixing. (Do not remove the bolt that the leg hinges around)
- 4) Swing out the legs into the extended position. Refix the four removed bolts.
- 5) Tighten all eight bolts that hold the legs. Lower the bed into position.

Using your bed: Do's & Don'ts

- DO** ensure you read these instructions and fully understand them before using your table.
- DO** ensure a copy of the "IMPORTANT Siderail safety notice" is included with these instructions.
- DO** ensure the bed is cleaned on a regular basis
- DO** ensure the bed is inspected on a regular basis (minimum once yearly service).

- DO NOT** exceed the safe working load of 180Kg (28 Stone)
- DO NOT** allow children to play with or under the bed
- DO NOT** allow any part of the bed or mattress to come into contact with a naked flame or excessive heat sources.

Adjusting the bed

Bed ends

These slide into the appropriate channels situated at each end of the mattress frame. Removal of the bed ends will give all round patient access, should this be required.

Nursing backrest and foot end (Where fitted)

- Ensure the person can support themselves while the back rest is adjusted. If they are incapable of supporting themselves seek assistance while adjusting the backrest.
- Ensure the backrest mechanism is firmly engaged before placing weight on the backrest.



Operate the backrest by the button situated to the rear of the "lock box" on the top of the sliding backrest. Pushing the unit down the slide rods swings out the backrest out into the required angle. Releasing the button locks the backrest into position.

To raise the backrest simply lift the "lock box" upwards.

Care and maintenance

Biocote



The painted parts of your table come with an antibacterial additive incorporated into the powder coated metalwork. Biocote™ inhibits the growth of bacteria on the surface of the table frame. It is effective against a wide range of both gram positive and negative bacteria as well as fungi. Examples of bacteria that Biocote™ is resistant to include:

Bacillus Subtilis
Escherichia Coli 0157
Salmonella Enteritides

Staphylococcus Aureus (MRSA)
Streptococcus Faecalis
Listeria Monocytogenes



Biocote™ is an aid to keeping your bed infection free but it does not substitute the need for the bed to be cleaned at regular intervals.

Cleaning

All surfaces to be wiped down with a disposable cleaning cloth, moistened with a mild detergent and diluted in hot water (60°C). Extra care should be taken around areas where excess dirt or dust may gather. Rinse down with clean water and dry off with a paper towel.

Always ensure that the bed is dry before re-using.

It is not recommended to use any bleach or surface cleaner that may scour the painted surface.

Inspection

The Boston Home Care bed requires very little maintenance, however if there are any signs of damage, of the bed is not performing as it should, please contact your supplier of assistance. Only authorised Sidhil service or repair agents should carry out repairs or service activities. Failure to do so may result in the manufacturer's warranty becoming void.

- Check that the maximum loading has not been exceeded
- Check the eight bolts holding the legs are tight
- Check there are no loose parts or fixings
- Check no parts appear excessively worn

Accessories

1292/I/W Nursing backrest and foot end - Adjustable angle backrest and removable footrest.

3004 – Backrest – Five position, easy clean, low cost backrest.

1292/6/LP Lifting Pole - Magnolia lifting pole. May be fitted to the left or right side of the bed.

1292/7 Bed Extension - Additional 150mm extension.

1210/DP Drip Pole - Two hook, adjustable height.

Side rails and Mattresses. - Contact Sidhill customer services to discuss the current ranges of these accessories. Both static and dynamic mattresses are available.

Warranty

Sidhil Limited standard terms and conditions apply to all purchases; a copy is available from the address below. Your statutory rights are not affected.



Sidhil Limited, Sidhil Business Park, Holmfield, Halifax, West Yorkshire HX2 9TN

Tel: +44 (0)1422 233030 Fax: +44 (0)1422 233040 Email: info@sidhil.com
Sales Tel: +44 (0)1422 233030 Sales Fax: +44 (0)1422 233040

A Member of the Siddall & Hilton Ltd Group of Companies



INSTRUCT-1202-5-PF-PF rev0
07/08/06