

**TAiMA** **M GT** **S GT** **TAiMA** **M ECO**

## INTRUDUCTION

The TAiMA rollator is exceptionally light, strong and can easily be folded to stow away. It provides ideal support during all your daily ventures as well a seat to rest.

Before using the rollator for the first time please read this manual carefully and make sure to be familiar with all features of the product.

The rollator described on this information sheet complies with EU Directive 93/42 for medical products. These instructions for use are an integral part of the scope of delivery.

The user should keep them available at all times and keep them with the product if this is passed on to a third party.

The manufacturer reserves the right to make changes relative to the information provided in these instructions as a result of technical developments and improvements.

This document may be reprinted, translated and/or duplicated in any form whatsoever, in part or in full, only subject to the manufacturer's prior written consent.

There is no update service for this users manual. Please contact the manufacturer to keep abreast of new developments.

## SPECIFICATIONS AND INTENDED PURPOSE

The TAiMA rollator is intended as a walking aid for persons with impaired mobility who weigh up to 150 kg. It also provides a limited seating function for temporary rest.

The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose – in particular as a climbing aid or to carry heavy loads. Any such use can be dangerous!

## CONTRAINDICATION

Use of the rollator is unsuitable in case of

- False sensation
- Severe disequilibrium
- With a body weight above 150 kg

## SAFETY INFORMATION

Do not hang bags on the handles as this could cause the rollator to tip over.

Use the service brake (by pulling the brake lever up) on inclines.

Only use the rollator on firm, flat surfaces.

Do not lean too heavily on the handles. Otherwise, the rollator could tip over backwards.

Be careful not to trap your fingers or clothes when folding up the rollator.


Upholstered parts become hot when exposed to direct sunlight and can cause damage to the skin if you touch them. These parts should be covered over. Alternatively, keep your rollator away from direct sunlight.

Before using your rollator, make sure the locking pin is correctly engaged and the rollator can not fold up randomly.

Before each ride, check that the brakes are working properly and that all bolted connections are tight.

Always lock both brakes before sitting on your rollator.

The maximum load for the shopping bag is 5 kg.

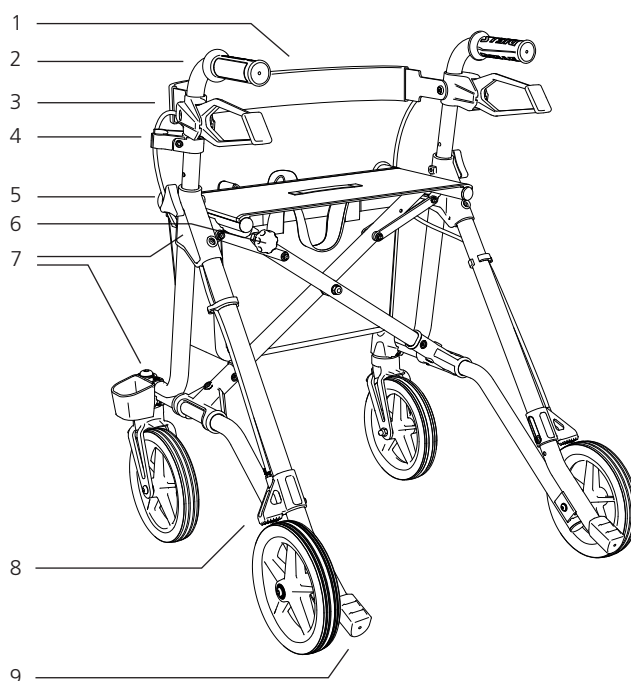


1. Backrest
2. Ergonomic handgrips
3. Brake lever
4. Cane holder
5. Height adjustment screw
6. Locking pin
7. Reflectors
8. Brake
9. Curb climber

## SETTING UP / UNFOLDING YOUR ROLLATOR

Place the rollator in front of you in the direction of travel.

Using both hands, press the tubes on either side of the seat down to the left and right until the locking pin **A** clicks into place.


**Caution!** The rollator must never be used unless the locking pin **A** is properly engaged.

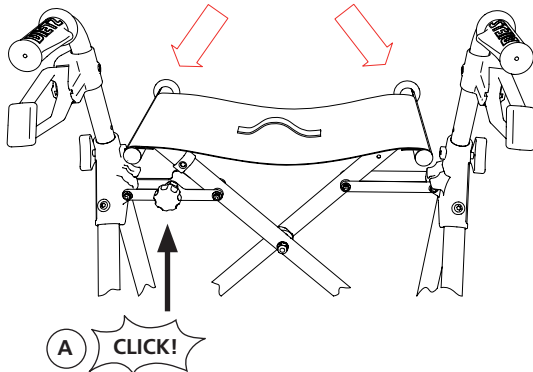

## FOLDING UP YOUR ROLLATOR

Release the locking pin **A** by pulling it slightly backwards and pull up the seat by its folding strap.

The rollator will now fold up automatically.

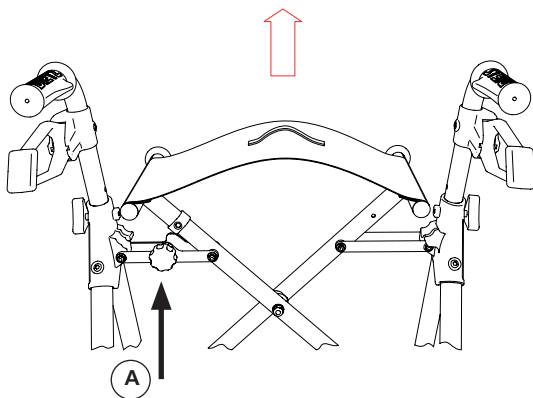


## GRIP HANDLE ADJUSTMENT

Loosen screw **B** and remove it completely.

Adjust the grip handles to the height desired.


Place screw **B** back in its hole and tighten it firmly.

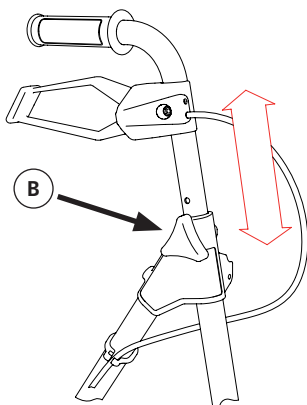

## BRAKE ADJUSTMENT

The brake is set up correctly if there is no play in either the brake lever or the Bowden cable and if the wheels turn without any rubbing or grating.

The braking power can be adjusted using the set screw **C** of the Bowden cable on the brake blocks of the back wheels.

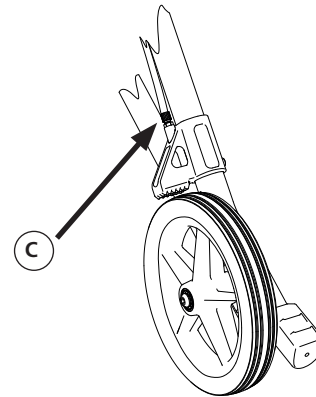


## WALKING WITH YOUR ROLLATOR

Position yourself inside the rollator. Grab the handgrips with both hands. As soon as you stand safely, you can loosen the parking brakes.


- Do not push the rollator too far in front of you. In order to give good support, the rollator should be kept close to your body.
- Control your speed by gently applying both brakes evenly.
- Walk upright and look forwards to be able to avoid any upcoming obstacles early. Walking backwards with your rollator is difficult, and not recommended.

## BRAKING

To reduce speed while walking just pull both levers evenly and gently upwards. Always apply the same amount of power on both brake levers.

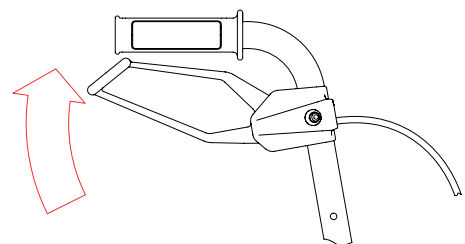


To lock the brakes, push the levers downwards. The brake levers will lock audibly into place. To release the brakes just pull the lever upwards again.

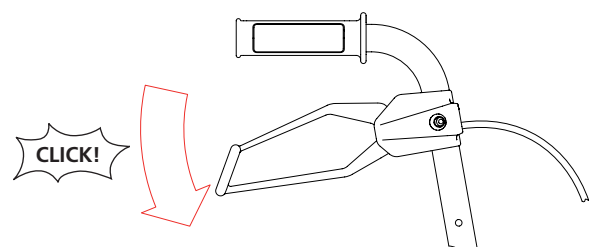

## CURB CLIMBER

To make climbing curbs easier you can lift the front wheels by pushing the rubber cap at the very end of the rollator frame downwards with your foot. Make sure to stand securely before using the curb climber.

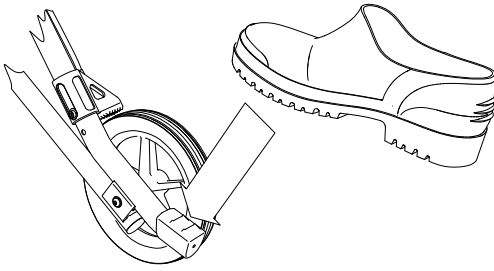

## TECHNICAL DATA

	TAiMA S_GT	TAiMA M_GT	TAiMA M_ECO
Height	76 - 86 cm	79 - 95 cm	
Height folded	76 cm	81 cm	
Seat height	51 cm	59 cm	
Seat width	39 cm	44 cm	
Length	62 cm	68 cm	
Width	58 cm	63 cm	
Width folded	23 cm		
Max. user weight	150 kg		
Weight with std. accessories	5,6 kg	5,8 kg	8,1 kg
Weight without accessories	5,3 kg	5,5 kg	7,8 kg
Frame material	Aluminium		Steel
Tyre dimensions	20 x 3,5 cm		
Grip handle adjustment	5 x 2,5 cm		
Max. load shopping bag	5 kg		

## SCOPE OF DELIVERY

TAiMA\_S\_GT, TAiMA\_M\_GT & TAiMA\_M\_ECO

- Rollator
- Backrest
- Cane holder left or right use
- String bag
- User manual

## CLEANING / MAINTENANCE / REUSE

### Cleaning

Clean and disinfect the rollator at regular intervals. To do so, use mild soapy water with no scouring additives. You can disinfect the rollator using commercially available products. Never use steam degreasers. Never use scouring or caustic cleaning agents.

### Maintenance

The TAiMA rollator requires no maintenance. If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the rollator to your specialist dealer for a service.

### Reuse by third parties

If you pass on your rollator to a new user, please remember also to hand over all the technical documentation needed to ensure safe operation. In accordance with specifications, the rollator must first be inspected, cleaned and returned to pristine condition by a specialist dealer.

## WARRANTY

The TAiMA rollator has a five-year warranty on the aluminum frame and a 24-month warranty on all other parts. Parts subject to wear or are excluded from warranty. Warranty does not cover any sort of misuse of your rollator.

## ACCESSORIES (OPTIONAL)

Backrest „Comfort“ soft, extra wide  
Art. 290038

Shopping bag for rollator  
Art. 290101 (M-GT & M\_ECO)  
Art. 290102 (S-GT)

One hand brake system, left or right,  
Art. 290105 / 290104

Extended push handles (only for TAiMA M-GT & M\_ECO)  
Art. 290106

Umbrella to protect from sun or rain incl. mount  
Art. 601270 (red)  
Art. 601271 (macchiato)

Hood, grey  
Art. 297002



Dietz GmbH Reha-Produkte  
Reutäckerstrasse 12  
76307 Karlsbad Ittersbach  
Fax: +49 (0) 7248 / 9186 - 86  
Fon: +49 (0) 7248 / 9186 - 0  
www.dietz-reha.com

