drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

all purpose stool
with adjustable arms

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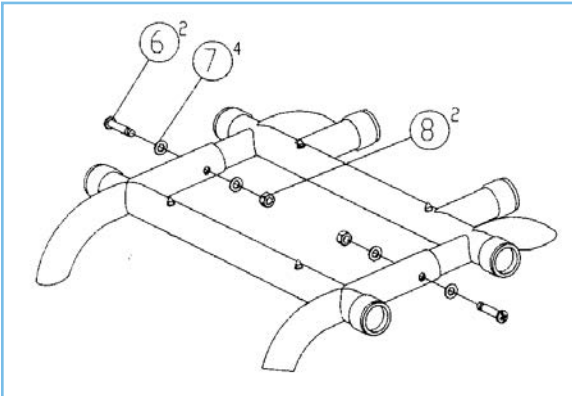
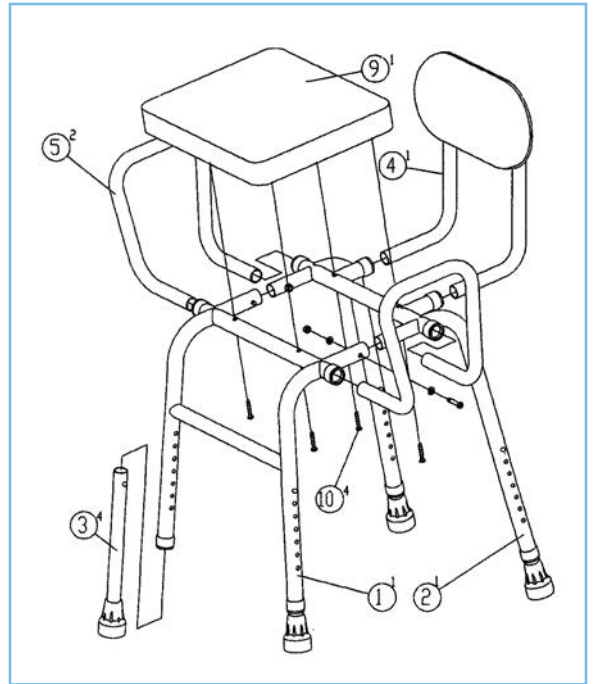
for use with item # 12455

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

assembly instructions

1. Secure part #1 (Front frame) to part #2 (Back frame). Insert part # 3 (Foot pieces) into the Front and Back frame. Insure when installing part #3 (Foot pieces) into the frame, both buttons protrude through the notches in the Frame.
2. Insert part #4 (Backrest) into the Back frame and insert part #5 (Handle) into the Front/Back frame.
3. Use parts #6, #7 and #8 to tighten the Front/ Back frame.
4. Put part #9 (Seat) and insert with part #10 (Screws) to tighten the Seat.



WEIGHT CAPACITY 300 LBS.

drive
MEDICAL DESIGN & MANUFACTURING

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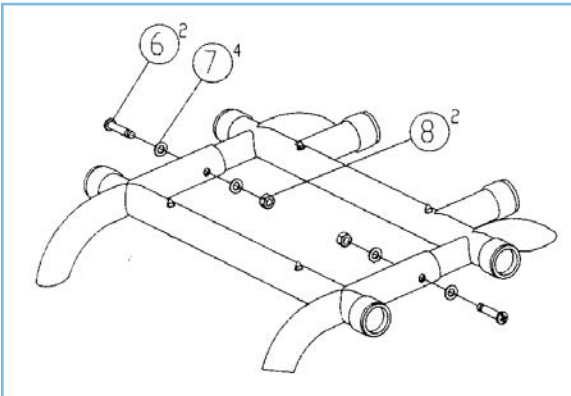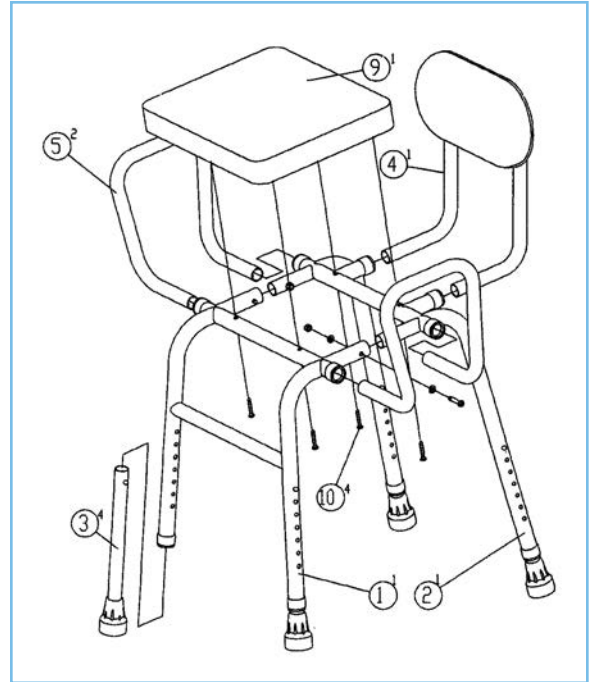drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

CAPACITÉ DE LA CHAISE: 136 KG.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.