

ComfortCushion is a firm cushion in anti-slip material that is placed under the patient's knees to prevent the patient from sliding forward when sitting up in bed. While the patient is prevented from sliding forward, the cushion also prevents stretching at the back of the thighs, which can otherwise occur when the legs are straight. ComfortCushion is made of a material with closed cells, which makes it light and easy to keep clean.

ComfortCushion is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

RoMedic's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.



Functional inspection

Visual inspection



Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.



Always read the instructions

Read instructions for all assistive devices used in connection with transfers.
To prevent discomfort, exercise care and caution when using the product.

Placement of the ComfortCushion:

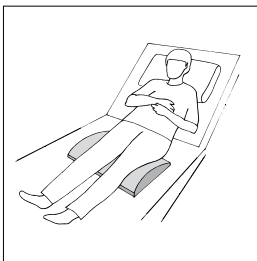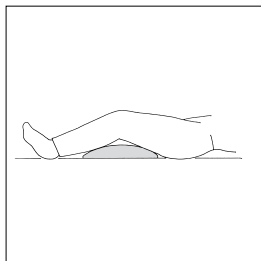

Placed under the patient's knees, when the patient is lying down, to prevent the patient from sliding downward when the head end of the bed is raised.

When raising the head end of the bed, ensure that pressure against the shoulders is alleviated, either by leaning the patient forward or by placing a friction-reducing device behind the shoulders.

NOTE! There may be a risk for contractures. Do not leave the cushion under the knees for too long.

Material:

Polyethylen

Care of the product:

Clean with dishwashing liquid or disinfectant. Wipe with a cloth.

Note: Do not subject the product to hard pressure or impact. This may leave indentations in the material.

When in doubt, contact your RoMedic representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative, or visit www.romedic.com. Make sure you always have the most recent version of the manual. Manuals are available for downloading from our website;

<http://www.romedic.com>

RoMedic AB

Box 640

SE-175 27 Järfälla

SWEDEN

Tel: +46 (0)8-594 773 33

Fax: +46 (0)8-594 773 44

Internet: www.romedic.com

E-mail: office@romedic.com

RoMedic[®]
A company in the Handicare Group