Intended Uses

Commodes allow for, or increase, independence when requiring the toilet, particularly by compensating for impaired movement or disability.

Description

This robust commode is made from epoxy-coated steel and folds easily for storage and transportation. The backrest and toilet seat are upholstered to provide additional comfort for the user. Both seat and pan come complete with lids for hygiene.

Instructions for use

1. Unfold the commode by gently pulling the rear legs away from the main frame. Ensure the commode is fully unfolded and stable before use.
2. Slide the commode pan into the rails under the seat and place the seat cover over the toilet seat.
3. Make sure the armrests are folded down for support.
4. If required, fold the armrests up for sideways transfers and for storage.

Cleaning and care

Clean frame with mild non-abrasive cleanser and water, then towel dry. Clean seat with mild disinfectant, then towel dry. Clean pan after each use with mild disinfectant, then towel dry.

Maximum user weight **133 kg** **292 lbs** **20³/₄ st**

Cautions

- The commode is not to be used to support more than 133kg / 292lbs / 20³/₄st
- Ensure the commode is suitable for the user
- Ensure the commode is fully unfolded and stable before each use
- Under no circumstance should the commode be moved whilst occupied

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy. Please retain these instructions for future reference.

Performance Health International Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
Tel: +44 1623 448 706
www.performancehealth.co.uk

 Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France

Distributed in the United States By:

Performance Health Supply, LLC
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
+ 1800-323-5547
www.performancehealth.com

Distributed in Australia By:

Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Pemulwuy NSW, 2145, Australia
1300 473 422
www.performhealth.com.au

Distribué en France par:

Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France
00(33) 03 10 00 79 30
www.performancehealth.fr

EC REP

CEpartner4U
Esdoomlaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.com

MD UK CA CE

 **PERFORMANCE HEALTH**

Intended Uses

Commodes allow for, or increase, independence when requiring the toilet, particularly by compensating for impaired movement or disability.

Description

This robust commode is made from epoxy-coated steel and folds easily for storage and transportation. The backrest and toilet seat are upholstered to provide additional comfort for the user. Both seat and pan come complete with lids for hygiene.

Instructions for use

1. Unfold the commode by gently pulling the rear legs away from the main frame. Ensure the commode is fully unfolded and stable before use.
2. Slide the commode pan into the rails under the seat and place the seat cover over the toilet seat.
3. Make sure the armrests are folded down for support.
4. If required, fold the armrests up for sideways transfers and for storage.

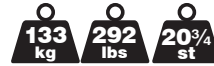
Cleaning and care

Clean frame with mild non-abrasive cleanser and water, then towel dry. Clean seat with mild disinfectant, then towel dry. Clean pan after each use with mild disinfectant, then towel dry.

Cautions

- The commode is not to be used to support more than 133kg / 292lbs / 20³/₄st
- Ensure the commode is suitable for the user
- Ensure the commode is fully unfolded and stable before each use
- Under no circumstance should the commode be moved whilst occupied

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