

# 12" Grab Rail - Fitting and User Instructions

MI0233 issue 4


(Model: AB0001; AB0007 & AB0025)

Congratulations on purchasing your 12" Grab Rail by Mangar International. The Grab Rail offers a safe hand hold providing;

- a support to push or pull against when standing up or sitting down.
- a steadying support while sitting down.
- a firm grip whilst transferring from one position to another.
- balance when standing or dressing.

## Contents:

- 1 x 12" Grab Rail.
- 6 x wall fixing screws.
- 6 x wall plugs.

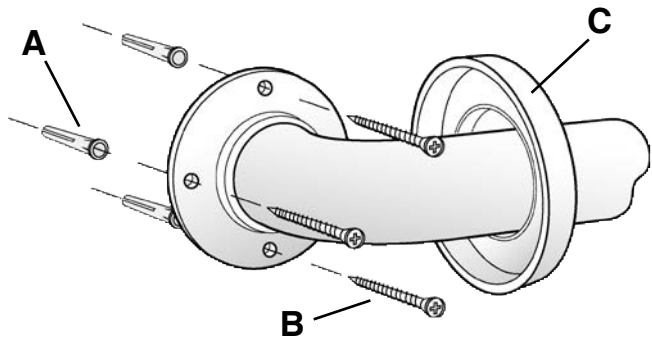

## Fitting

**WARNING:** Before fitting, ensure that there are no concealed electrical cables or pipes where the fixing holes are to be drilled.


1. The fixings supplied are ONLY suitable for use in solid walls. If fitting to plasterboard walls ensure suitable fixings are used. If in doubt consult a qualified tradesman.
2. Position the Grab Rail on the wall either horizontally or at an angle to suit and mark the six hole locations.
3. Drill the six holes to a depth approximately 5 mm greater than the length of the wall plugs provided and insert the plugs (**A**).
4. Fit the screws (**B**) through the holes of the base and into the plugs and fully tighten.
5. Push the covers (**C**) onto each base to conceal the screws and complete the fitting.

---

## Using the Grab Rail

- Before first use, check that the Grab Rail is securely fitted to the wall.
- Use the Grab Rail to ASSIST when lowering or raising yourself. NEVER apply your full weight to the Grab Rail.
- Regularly clean the Grab Rail with a non-abrasive proprietary cleaner.

## Specifications:

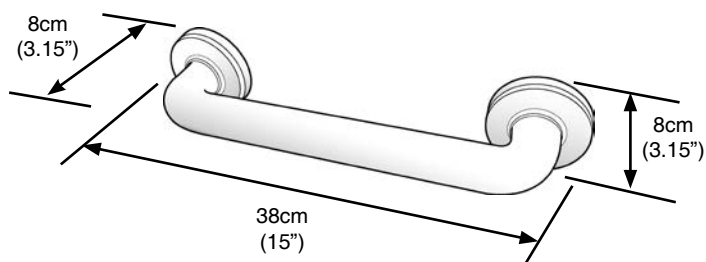

## Warranty

Mangar International guarantees this product for a period of one year.

The guarantee is given against defects in the product. The guarantee is not valid in the case of fair wear and tear, wilful damage, common neglect, misuse or unauthorised alteration or repair by unqualified persons.

The following statements do not affect the consumer's statutory rights. This product is sold on the understanding that in the event of any defect of manufacture or material appearing within one year of the date of receipt of this product, the product will be repaired or replaced free of charge providing that:

- Reasonable evidence is provided (e.g. purchase invoice, delivery note) that the product was not purchased more than one year prior to the date of the claim.
- The defect is not attributable to accidental damage (either in transit or otherwise), misuse or unauthorised repair.

Mangar International reserves the right to alter product specifications and/or any of the information contained within this document without notice.



**Mangar International**

Presteigne, Powys, Wales, UK, LD8 2UF

Tel: +44 (0)1544 267674

Fax: +44 (0)1544 260287

customercare@mangar.co.uk

www.mangarinternational.co.uk

**Mangar**  
INTERNATIONAL