



IncoSheet is a washable incontinence barrier that can be used in bed or on a wheelchair seat cushion. The material is highly absorbent and distributes fluid over a large area. The fabric on the top side of the product makes the surface feel dry and comfortable. The rough fabric on the underside provides friction to keep the sheet in place in the bed.

IncoSheet is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

RoMedic's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.



Functional inspection

Visual inspection

Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.



Always read the instructions

Read instructions for all assistive devices used in connection with transfers. To prevent discomfort, exercise care and caution when using the product.

Placement of IncoSheet



75x90 cm


Place the IncoSheet with the quilted, lighter side facing upwards in the bed. IncoSheet is available in four different sizes: two sizes have a shorter incontinence barrier, of which one has fabric on both sides so that it can be bedded in under the mattress, and one size with a longer incontinence barrier. A smaller size, specially developed for placement in a wheelchair, is also available.

The two shorter incontinence barriers are mainly intended for patients who are able to turn themselves, but they can also be used when the patient requires assistance to turn, in which case a transfer assistive device is placed on top. Since the transfer assistive device is placed on top of the IncoSheet, the IncoSheet remains in the same place under the patient during and after turning, thus continuing to provide incontinence protection.



75x200 cm

IncoSheet with the longer incontinence barrier is mainly intended for patients who need help with turning. This size model can be placed on top of the transfer assistive device, since it has an extra long incontinence barrier and continues to provide protection, even after turning.

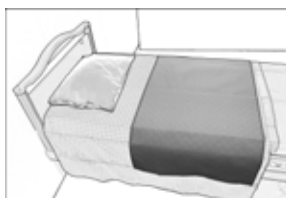

75x150 cm

The small model, IncoSheet 40x83, can be placed in a wheelchair to protect the seat cushion.



40x83 cm

Note!

IncoSheet is not designed to be used as a draw sheet, but is instead bedded over a bed sheet, a draw sheet or other transfer assistive device.

Material:

Polyester, cotton

Care of the product:

Read the product label.



IncoSheet can be laundered and dried at high temperatures. Do not use laundry rinse or bleach (e.g., chlorine). IncoSheet withstands more than 100 launderings.

Size:

1530	75 x 90
1531	75 x 200 with fabric on both sides
1532	75 x 150
1533	40 x 83

When in doubt, contact your RoMedic representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative, or visit www.romedic.com. Make sure you always have the most recent version of the manual. Manuals are available for downloading from our website;

<http://www.romedic.com>

RoMedic AB

Box 640

SE-175 27 Järfälla

SWEDEN

Tel: +46 (0)8-594 773 33

Fax: +46 (0)8-594 773 44

Internet: www.romedic.com

E-mail: office@romedic.com

RoMedic[®]
A company in the Handicare Group