

N.B. This equipment must be installed by a competent installer/ technician.

VR127/VR128/VR129	Medina Bath Seat	30st (190 kg) weight limit
VR116/VR117/VR118	Medina Bath Board	25st (154kg) weight limit
VR124/VR125/VR126	Medina Shower Board	25st (154kg) weight limit

Do not exceed the weight limit stated – doing so could put the user at risk.



VR127/VR128/VR129



VR124/VR125/VR126



VR116/VR117/VR118

1. All Bath Seats, Bath Boards and Shower Boards


Please examine your product for any visible damage before using/ installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

2. Cleaning

Only use a mild detergent or disinfectant, never use an abrasive cleaner. You may autoclave all Medina products up to a temperature of 85°C. Always ensure that you wipe dry all equipment and treat joints with a water dispersing agent, e.g. WD40 after washing.

3. Fitting Bath and Shower Boards

- 3.1 Turn the board over with the cleats facing upwards (see fig. 1)

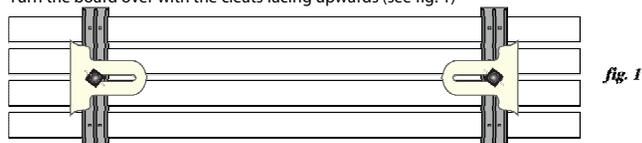

- 3.2 Loosen off the wing nuts holding the wing nuts holding the cleats in position (see fig. 2)
 3.3 Turn the board over so that the slats are facing upwards.
 3.4 Place the board over the bath traversing its width.
 3.5 Reach underneath the board and push both cleats outwards evenly until they are gripping the wall of the bath.
 3.6 Tighten off the wing nuts. N.B. It is only necessary to tighten the wing nuts finger tight. Do not overtighten them as this can cause the threaded brass insert to come loose, which could cause the product to fail.
 3.7 Double check the cleats are friction tight to the wall of the bath before use.

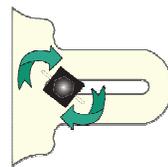


fig. 2

Always check the tightness of the board before each use

Always repeat the installation every time the board is removed

Always make sure the correct width of board is installed to suit the bath onto which it is to be fitted.

4. Fitting Bath Seats

- 4.1. Firstly, ensure that the base of the bath is free from grease or any form of obstruction and is perfectly clean.
 4.2. Take the bath seat and dampen the four suction pads on the underside of the seat (see fig. 3).

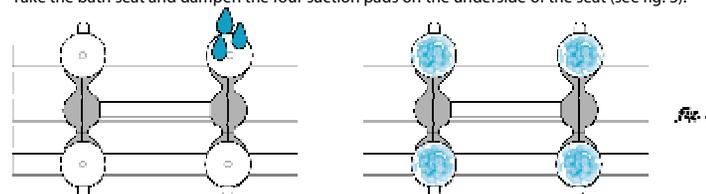


fig. 3

- 4.3. Place the bath seat onto the base of the bath (normally at the opposite end to the taps) with the slats of the seat traversing the width of the bath.
 4.4. Push down firmly on the flaps ensuring that all four flaps make full contact with the base of the bath (see fig. 4)

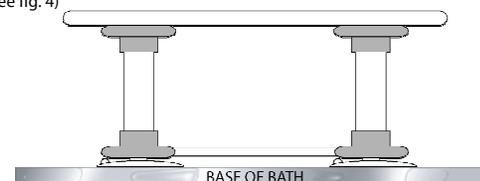


fig. 4

- 4.5. Check that the bath seat is secure by trying to lift the seat without disengaging the flaps



Always check the tightness of the board before each use

Always repeat the installation every time the board is removed

Should you wish to remove the bath seat for any reason, this can be done by simply lifting the ears of the 4 suction caps (see fig. 5), this will cause the suction to be released. To refit, repeat the fitting instructions above.

5. Cleaning

The Medina range of Bath Seats should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth only. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

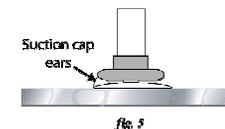


fig. 5

6. Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes:

- Tightness of all nuts/bolts/casters and other screw in/bolt in/push in components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

7. Important Information

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or Aidapt Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

8. Maintenance

Please make a safety check of the product at regular intervals or if you have any concerns.