

User Guide


UNIVERSAL Side Rails

SIDHIL


Thank you for purchasing a pair of Sidhil Universal Side Rails. These user instructions cover the fitting and operation of the products below:

UNIVERSAL/SR/MAG: Standard Height Magnolia
UNIVERSAL/SR/WH: Standard Height White

UNIVERSAL/SR/HI/MAG: Extended Height Magnolia
UNIVERSAL/SR/HI/WH: Extended Height White


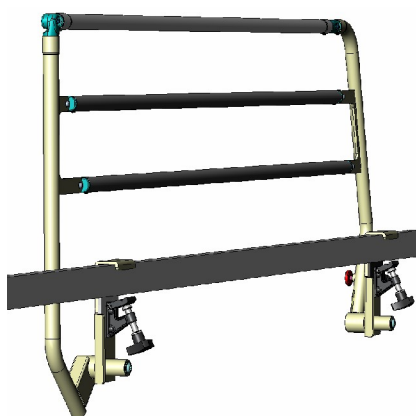


Standard Height

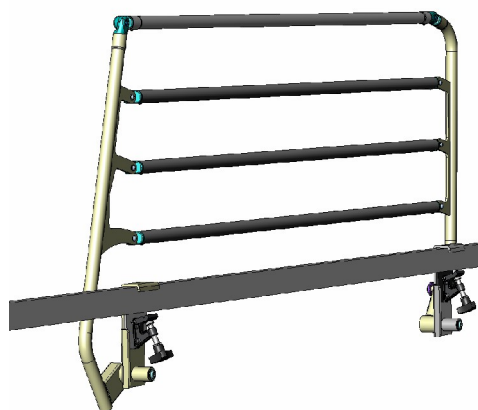


Extended Height

Assembly

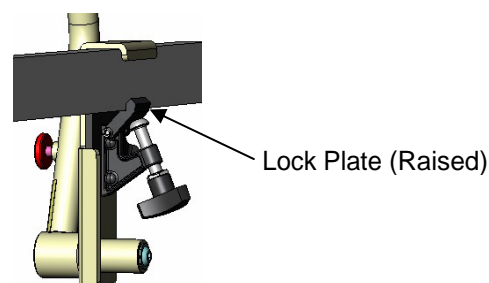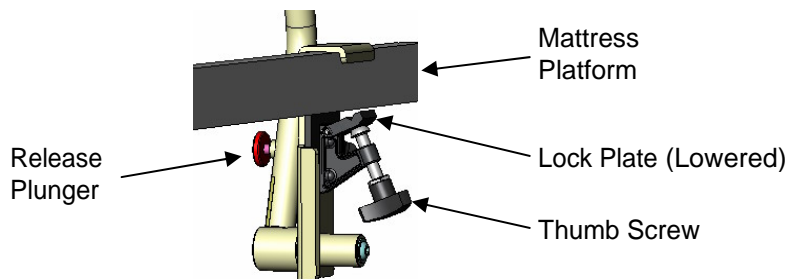

1. Orientate the side rails so that the Clamp-on-Bracket with the red release plunger is at the foot end of the bed and the release plunger is pointing outwards.
2. Unscrew the Thumb Screws on each Clamp-on-Bracket to lower the Lock Plate.
3. Hook the two Clamp-on-Brackets over the side of the mattress platform. The Lock Plate must remain underneath the mattress platform.
4. Position the Side Rail along the side of the bed. (Refer to MHRA Device Bulletin DB2006(06) December 2006)
5. Tighten the Thumb Screws on each Clamp-on-Bracket to raise the lock plate firmly against the mattress platform
6. Repeat the above procedure for the second side rail.
7. The side rails are now assembled and ready for use.

Operation

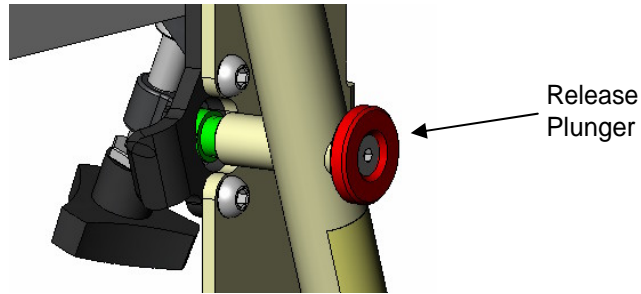

- Lowering the side rail:
To release the side rail pull the spring loaded Release Plunger outwards. Whilst holding out, gently lower the side rail.
- Raising the side rail:
Gently lift the side rail upwards into position. The side rail will automatically lock once fully raised.

Cleaning

Wipe down with a slightly moistened cloth using a mild detergent. Dry thoroughly. It is not recommended that bleach is used as it may cause marking to the frame.

Maintenance

- Check periodically that the Thumb Screws are tight and the Lock Plates are firmly engaged.
- Sidhil recommend that the side rails are serviced once annually as a minimum.

If you are in any doubt about the suitability or fitting of the side rails always seek expert advice

Sidhil Limited Sidhil Business Park Holmfield Halifax West Yorkshire HX2 9TN

**Tel: +44 (0)1422 233030 Fax: +44 (0)1422 233040 Email: info@sidhil.com
Sales Tel: +44 (0)1422 233030 Sales Fax: +44 (0)1422 233040**

A Member of the Siddall & Hilton Ltd Group of Companies