

WEBBING LADDER

This multi-coloured webbing ladder is designed for:

- Encouraging patients to sit themselves up in bed/ lower themselves into bed in a controlled way
- Safe usage—the spaces between the rungs are small so that there is no danger of patients inserting their heads through them
- Suitability for patients of all heights, with 10 rungs allowing for maximum durability

In addition:


- Easy to use
- Safe
- Strong
- No knots to be tied so easier for staff

Instructions for use:

1. Thread the large loop up through the end of the bed or part of the bed frame and hold it in an "O" shape
2. Thread the rest of the ladder through this "O" and pull tight
3. The ladder is then securely held

Washing Guidelines:

- Wash at 74°C
- Drip dry, or tumble dry on low heat
- Do not iron
- Do not use chlorine bleach
- Can be autoclaved

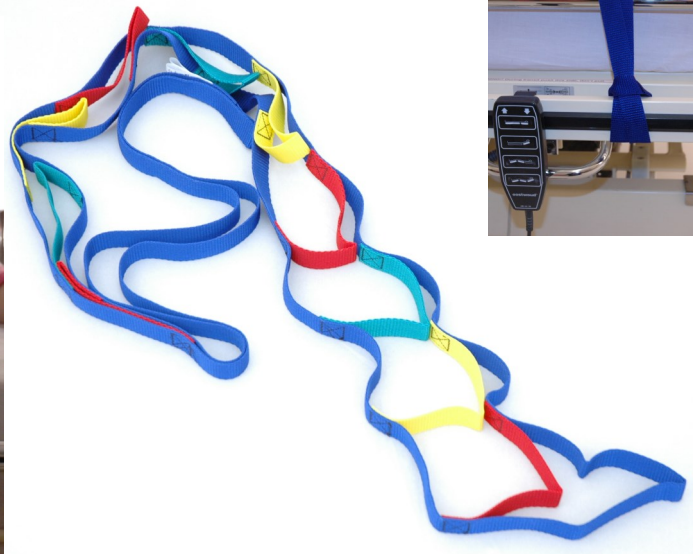

Hospital Direct

Units 3 & 4, The Green, Clun, Shropshire SY7 8LG
Tel: +44 (0)1588 640144 Fax: +44 (0)1588 640661

www.patient-handling.com